

THE EXPERT'S GUIDE TO
**DEPROGRAMMED
BITE RECORDS**

Gold Dust's step-by-step guide for bringing cosmetic dentistry to the next level.



WHAT DO YOU GET?

- A guide to taking full mouth rehabilitation deprogrammed bite records.
- A step by step outline of how the appointment process should proceed.
- The same protocols taught at the Clinical Mastery Series.



As developed by the Clinical Mastery Series protocol, the following is a guideline for taking a deprogrammed bite records for your diagnostic wax up. This appointment in general, follows the delivery of the patient's release appliance and protocol.

01 Take composite ball record. Keep patient from closing on back teeth once release appliance is removed. Add composite in between maxillary and mandibular centrals, just enough to keep back teeth from touching.

02 Determine final vertical dimension of the case. (CEJ to CEJ measure)

- If no vertical change then is this a CO case?
- Full intercuspation for CO
- If change in VDO then it is a CR case.

Take molar bite relationships with Futar D Fast Set

03 Take tripod bite using new anterior ball at new VDO

04 Gross Mock Up

- Central Length
- Smile Line
- Midline
- Buccal Corridor

05 Take Photos of Mock Up

06 Take Facebow record

07 Mark on pre-op model tissue reduction or indicate on prescription

08 Take impression of mockup

09 Take polyvinyl anterior bite with posterior bites in

10 Take stick bite with Rigistick

11 Mock up lower incisors and take impressions

12 Remove mock up and take necessary pre-op photos.

13 Complete Wax Up Rx

